Qualità e design firmato EME
Quality and design by EME

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Indications

The shock wave method is the chosen treatment for chronic insertional tendinopathies characterised by a poor vascularisation of the osteotendinous junction, where the physiotherapy treatment (infiltrations and laser therapies) has been proven ineffective.

These are the main pathologies that are suitable for shock wave application:

- Elbow – epicondylitis, epitrocleitis
- Shoulder - insertional tendinopathies, impingement
- Knee – patellar and "goose's foot" tendinopathies
- Pubis – adductor tendinopathies (pubalgia)
- Ankle - achilles tendinopathies, calcaneal apophysis

An example: rotator cuff calcific tendinitis

30 cases were examined of people suffering from this pathology who have been treated using the following shock wave therapy:

Pression: 3 Bar - Frequency: 10 Hz - Shock: 2500

Sessions: 4 (weekly)

Objective: elimination of calcific deposits

Conclusions: *The study has shown that, for 90% of treated patients the elimination of pain was total; while the remaining 10% experienced a considerable decrease in pain.*

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SHOCK MED - SW 1352

Device for radial shockwaves, power up to 1,5-5 Bar, adjustable frequency 1-20 Hz

SHOCK MED COMPACT - SW 1350

Portable device for radial shockwaves, power up to 1,5 - 4 bar, adjustable frequency 1-15 Hz

SHOCK MED and SHOCK MED COMPACT are certified devices
KIWA CERMET ITALIA Spa CE 0476, according to Directive 93/42/EEC

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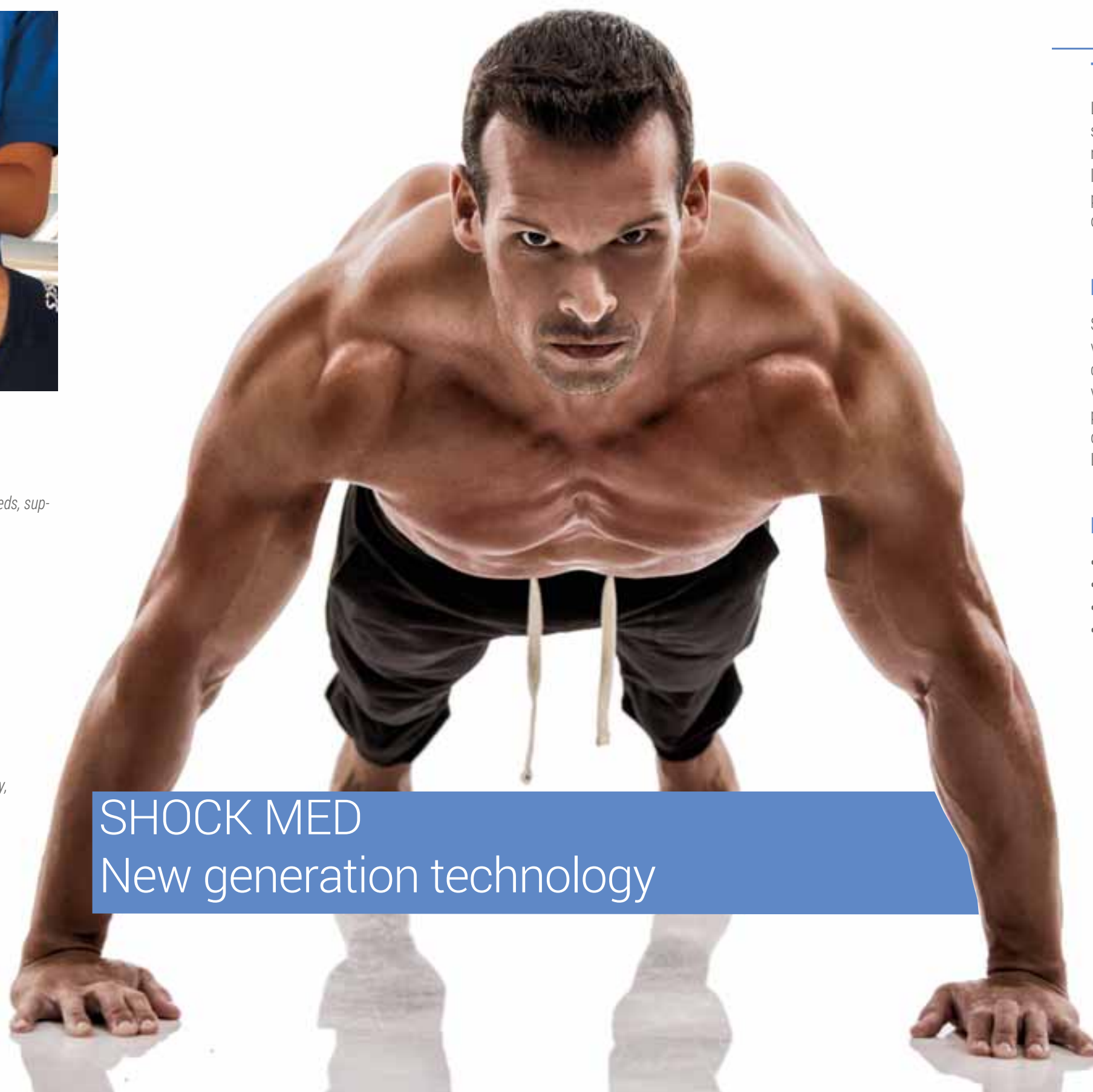
EME Physio presents: Shock Med

EME Physio worked out this project starting from the user's needs, supporting him in every phase of the therapy:

Selection of the work protocol in:

1. **Body area:** according to the body part the user will be able to select the most suitable therapy
2. **Pathology:** in this section all protocols to be used with Shock Med will be listed
3. **Patient file:** for storing all patient's data, treatments and short case history

One-Touch mode: by pressing just once the activation button the procedure will start and continue until the end of the therapy, or until the operator will press again the button to pause the execution of the treatment.

Procedure: a practical and quick guide for the application of the shock waves therapy, based on four fundamental phases:

- Localization by touch of the pain area
- Selection of the therapy criteria (manually or using default programs)
- Gel application
- Start of the shock waves therapy

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Cutting edge technology

Shock waves are acoustic waves transporting high energy transmitted through the skin surface and radiated into the body, to the painful area. The body reacts by increasing the metabolic activity in the application area, thus making it easier to minimize the inflammation caused by a pain-killing action induced by the local releasing of endorphins, thus stimulating and accelerating the recovery.

The shock wave

Shock Med is a radial (ballistic) shock wave, in the sense that the shock wave is generated by a special pistol-shaped probe equipped with a barrel which is plugged at one end by means of a metal cap which is projected using compressed air (up to 5 bar pressure), a steel bullet. Thus a shock wave is generated which is radiated into the skin and into the first tissue layer just below skin. Penetration depth may vary from 4 to 7 cm.

Main Shock Med applications:

- Orthopaedics
- Rehabilitation
- Sport medicine
- Aesthetic medicine