Qualità e design firmato EME  
Quality and design by EME

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## The advantages of Laser therapy

*Laser therapy is not based on the generation of heat, but on photochemical and photobiological effects on cells and tissues. Observations have shown that if the Laser light is supplied in the right quantities, you will obtain a stimulation of certain cellular functions, especially in the presence of cells with functional deficiencies. The biological action in using the Laser therapy produces a series of effects on the cells for a "stimulating" action on mitochondrial functions, with a higher production of ATP.*

**The applications of the Bi Power Lux laser produce a number of effects on the treated tissues:**

1. **Increase in hematic flon:** vasodilatation of capillaries and arteries.
2. **Biostimulation:** tissue regeneration, stimulation of the protein synthesis, stimulation of the production of ATP, stimulation of the fibroblast mitosis, increase in collagen and elastin.
3. **Anti-inflammatory effect**
4. **Anti-edematous effect:** stimulation of the lymphatic system.
5. **Analgesic effect:** increase in the perception threshold of nerve endings.

## Bi Power Lux the winning laser

**Powerful**, thanks to the high level of power supplied by its many versions. It allows the stimulation of the deepest layers of the treated tissue, thus favouring a rapid and diffused cellular regeneration. It is possible to obtain a deep tissue stimulation and this makes possible to treat the most internal tissues and structures (such as the femoral joint) and chronic pathologies such as arthrosis.

**Versatil**, thanks to its characteristics. Bi Power Lux may be used in many fields such as sports medicine, orthopaedics, neurology, dermatology, rheumatology and acupuncture.

**Effective** and crucial in acute, chronic and degenerative inflammations such as knee arthritis.

**Multispot**, Bi Power Lux automatically calculates the quantity of energy (fluence in joules) supplied in function of the different application modes, contact or through the 3 positions of the spacer included.

**Easy and safe to use** as all EME devices. Bi Power Lux has a software that can be used by all operators, with user-friendly controls that can be easily and rapidly used, designed to guarantee maximum safety standards.

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Technical specifications	BI POWER LUX 6W	BI POWER LUX 12W
Backlit LCD display	Graphic colour 8" Touch screen	Graphic colour 8" Touch screen
Peak power in continuous mode	6W	12W
Configuration of the laser probe	Integral connection	Integral connection
Emission wavelength laser diode	808 + 980 nm	808 + 980 nm
DNRO (m)	2,26	2,986
Emission frequency	100 – 10000 Hz	100 – 10000 Hz
Pulsed mode	10 - 100%	10 - 100%
Output channels	1	1
Stored protocols	32	32
Storable protocols on the internal memory	200	200
Storable protocols on the USB key	•	•
Size	39 x 17,5 x 28 cm	39 x 17,5 x 28 cm
Weight	7,5 Kg	7,5 Kg

Accessori in dotazione / Supplied Accessories	BI POWER LUX 6W	BI POWER LUX 12W
Occhiali protezione laser / Laser protective goggles	1	1
Interlock 3 poli / Interlock 3 poles	1	1
Pedale / Pedal	1	1
DVD Welcome Pack (Manuale d'uso) / (User manual)	1	1

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
**BI POWER LUX** devices are certified  
KIWA CERMET ITALIA Spa CE 0476, according to Directive 93/42/EEC

**EME**  
ITALY

Medical & Aesthetic Technologies

EME srl

Via degli Abeti 88/1 - 61122 Pesaro (PU) - Italy - tel. +39 0721.400791 - fax +39 0721.26385  
www.eme-srl.com info@eme-srl.com

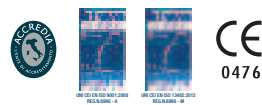E2706059

EME PHYSIO V4 - 10/2016 1000 EME COMMUNICATION

# BI POWER LUX

## HPLS High Power Laser System

**EME**  
ITALY  
physio

MADE IN ITALY